

SHAFER ENTERPRISES, LLC

**COOL SHIRT®**

*Personal Cooling Systems*

10 Andrew Drive Suite 200

Stockbridge, GA 30281


Sales & Service 800-345-3176 Fax 678-289-4325

www.coolshirt.net

# *Surgeon's* **COOL VEST** *Cart Handle*

## Installation Instructions

1. Select corner of cart desired for handle
2. Slip the metal rail sleeves of handle over top rails of cart as shown in illustration. Press down to ensure that they are well-fitted over the rail.
3. Place clamp located at bottom of handle over side rail of cart at the bottom corner.
4. Tighten nut & bolt on clamp to secure the handle to the cart.

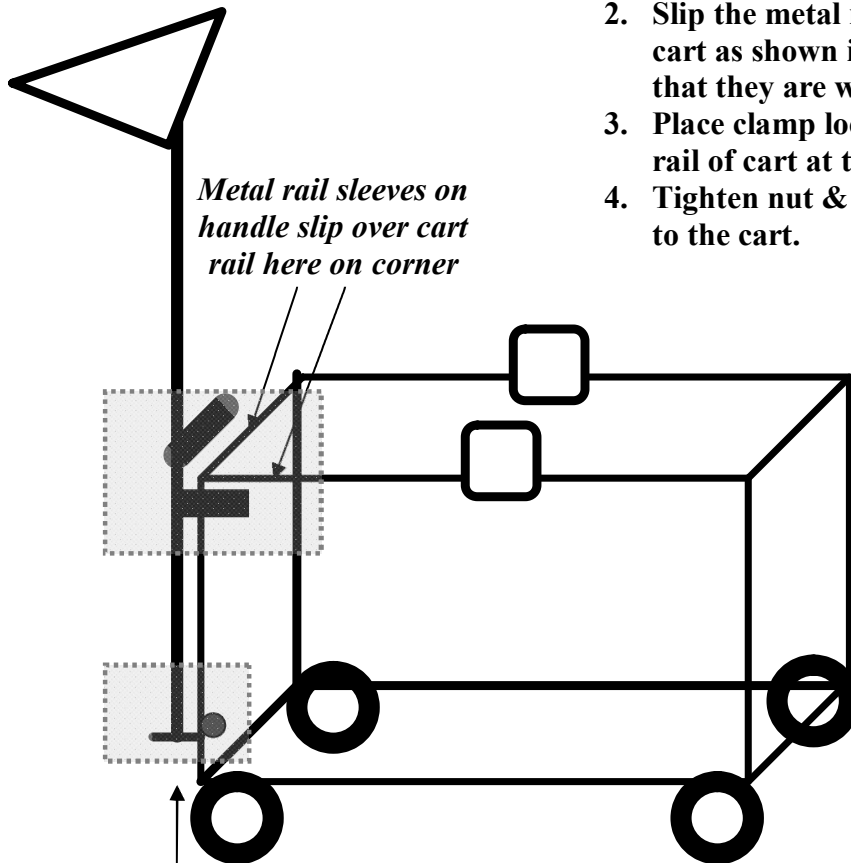

*Metal rail sleeves on handle slip over cart rail here on corner*

*Attach handle here with nut & bolt*

*Note: When installing on older carts, if handle is slightly angled after installation, gently bend it to meet your needs.*

Shafer Enterprises LLC 2006  
Cart Handle  
Part#CHDL-1